

GOFAST

REF EB02.064

Folding stretcher for evacuation and CPR.



Elite Bags S.L.
Vial de los cipreses 9
03006 Alicante



Rev.00 February 2024

1. Intended use

GOFAST is a tarpaulin folding stretcher that is manufactured for:

- Loading and transferring patients.
- Support in case of CPR.
- Facilitate the transfer of the patient from one stretcher to another.

2. User profile

The GOFAST stretcher is manufactured to be used by paramedics, fire brigades, rescue teams, civilian protection, military fields, ski patrol, lifeguards.

3. Product description

The GOFAST stretcher is manufactured with Tarpaulin 300D to be strong, flexible, resistant and waterproof. In addition, it is also resistant to bacteria, fungi, rot and oil. It is designed to be light, easy and portable.

Made with 11 handles to be used by 3 professionals. It has 3 independent pads in case of assistance and CPR to support the knees of professionals.

It is available in orange with the following measurements; width 70cm and length 190cm.

It has a carrying capacity of 150kg.

4. Instructions

1. Support the patient's neck.
2. Put the patient on their side or lift together.
3. Slide the Gofast under the patient.
4. Place the knee pads according to the image.
5. Assist the patient and/or perform CPR.
6. Grab the handles to lift and evacuate the patient.

WEIGHT LIMITATIONS DEPENDING ON TIME OF USE

WEIGHT	TIME
< 150Kg	Without limits
150Kg – 200Kg	Maximum 4h
200Kg - 250Kg	Maximum 20min.

5. Storage and reuse conditions

This device is reusable. It is important to disinfect and clean the stretcher after each use as well as all its components. Store once dry in its original case.








6. Contraindications/ warnings

1. BEFORE EACH USE check the condition and integrity of the device as well as all its components.
2. To ensure a secure grip, only use the handles to carry and transfer the patient.
3. Do not use if damage is observed to any component of the stretcher (inform the manufacturer).
4. To ensure safe loading and transportation, do not use by less than 3 professionals.
5. The device must not come into contact with heat sources or flammable agents.
6. Avoid contact of the device with sharp and pointed objects.
7. The use of the device for any activity other than its intended use is not permitted.
8. It is recommended to perform rescue drills with the device before use.
9. Before first use, check that the packaging is intact and includes all the components of the device as well as the instructions for use.
10. This device is not recommended when spinal stability is an issue.
- 11. Do not use on people weighing more than 250kg.**

ADVERTISEMENT:

Use only original Elite Bags S.L. components, never alter or modify the device. In this case, the manufacturer responsibility according to EU Regulation 2017/745 is declined, and the warranty is void.

Any serious incident that occurs in relation with this device must be reported to the manufacturer (calidad2@elitebags.es) and national competent authority (sgps@aemps.es).

						
Medical device	Catalogue number	Manufacturer	Device compliant with EU regulation 2017/745	Bath code	Unique Device Identifier	Consult instructions for use