

**OCCHIALI D'ISOLAMENTO MEDICALI**  
**MEDICAL ISOLATION GOGGLES**  
**LUNETTES MÉDICALES D'ISOLEMENT**  
**GAFAS DE AISLAMIENTO MÉDICO**  
**ÓCULOS DE ISOLAMENTO MÉDICO**  
**MEDIZINISCHE ISOLATIONSBRILLE**  
**ΙΑΤΡΙΚΑ ΓΥΑΛΙΑ ΑΠΟΜΟΝΩΣΗΣ**  
**MEDISCHE ISOLATIEBRIL**  
**MEDICINSK ISOLATIONSBRILLE**  
**MEDICINSK ISOLERINGSGLÖGONE**  
**LÄÄKETIETEELLISET ERISTYSLASIT**  
**MEDISINSKE ISOLASJONSBRILLE**  
**OKULARY IZOLACYJNE MEDYCZNE**  
**LÉKAŘSKÉ IZOLAČNÍ BRÝLE**  
**MEDICINSKA IZOLACIJSKA OČALA**  
**MEDICINSKE IZOLACIJSKE NAOČALE**  
**ORVOSI SZEMÉVEZET**  
**OCHEAREI DE IZOLAT MEDICAL**  
**МЕДИЦИНСКИ ИЗОЛАЦИОННИ ОЧИЛА**  
**ةي بطل لزعلا تاراظن**

**REF** GDG02 (Gima 25266)

**Zhejiang Gongdong Medical Technology Co., Ltd**  
No. 10 Beiyuan Ave., Huangyan Taizhou,  
318020 Zhejiang China  
www.chinagongdong.com  
Made in China



**ZOUSTECH S.L.**  
Pso. Castellana, 141- Planta 19, 28046-Madrid,  
Spain

Importato da / Imported by / Importé par / Importiert von/ Importado por/ Importado por/ Εισάγεται από / Importowane przez / Importat de / Importováno / Importerad av / Maahantuaja / Importerad av / Uvezao / Importálja / Importert av / Geïmporteerd door / Importeret af / Внесен от / داري تاسا :

**Gima S.p.A.**

Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)



**Instructions for Use**

Model: Fully Enclosed with Air Vents

Rev. A/1 Effective:2022-10-17

**Description**

The Medical isolation goggles is intended to create a physical barrier separating the eyes from the immediate surroundings. This device prevents liquids or droplet splash from contacting the user.

The Medical isolation goggles is intended for single use.

**Indication for Use:**

Medical isolation goggles are personal protective equipment intended for protection of the associated mucous membranes eyes from splashes, sprays, and spatter of body fluids.

**Contraindications:**

None.

**Product side effects:**

This product has no side effects.

**Warning:**

1. Do not use if the components show any sign of damage.
2. Do not expose components to temperature greater than listed on package labeling.
3. Dispose of the product as biohazardous waste per institutional protocol.

**Operation:**










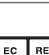
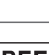
Take out the goggles from the packaging bag, confirm that the goggles in good condition, and tear off the protective film; the goggles will fix the elastic band behind the head, and adjust the soft fixation device to make it fit closely with the face.

**Storage:**

Store product in a dry place at 0 - 30°C.

Keep away from sunlight.

**Explanations for symbols**

	Expiration date		Disposable device, do not re-use
	Lot number		Don't use if package is damaged
	Date of manufacture		Keep away from sunlight
	Medical Device compliant with Regulation (EU) 2016/425		Keep in a cool, dry place
	Manufacturer		Authorized representative in the European Community
	Product code		

**GIMA WARRANTY TERMS**

The Gima 12-month standard B2B warranty applies

CE PPE Cat.II issued by:

DIN CERTCO Gesellschaft für Konformitätsbewertung mbH Notified Body  
 0196, Alboinstraße 56, 12103 Berlin, Germany  
 according to the Annex II of Regulation (EU)2016/425 - Harmonized EN  
 166: 2001 Directive 2001/95/EC